

A7066 - Foster's Enucleation Snare - Instructions for use.






For reprocessing and handling instructions, see the Altomed Reusable Instrument Instruction Sheet (Ref: ALT I013) available from the Quality Department at Altomed.



Important – Only use pliers made of stainless steel.

To assemble the Fosters Enucleation Snare:

1. Carefully bring both ends of the snare wire together. Do not fold the wire or cause it to bend or crease - Fig 1.
2. Carefully feed both ends of the snare wire through the barrel of the handpiece and through the retaining barrel on the securing handle - Fig 2.
3. Using a pair of stainless steel pliers carefully put a 90° bend in the last 5mm of each end of the snare wire - Fig 3.
4. Depress the retaining clip and rotate the securing handle 360° then allow the retaining clip to lock the handle in place – Fig 4.
5. The device is ready for use.

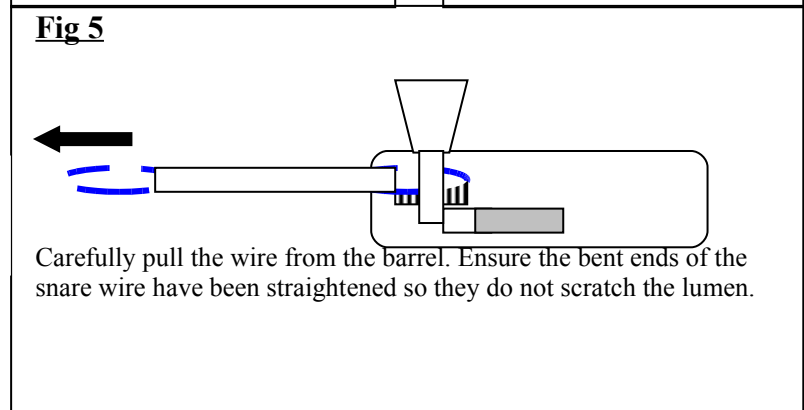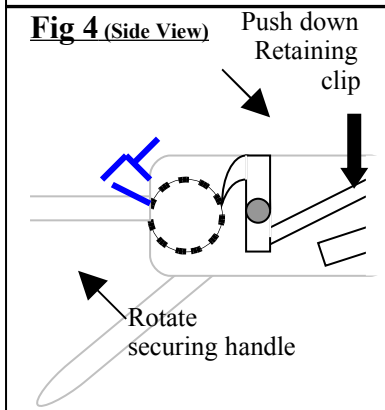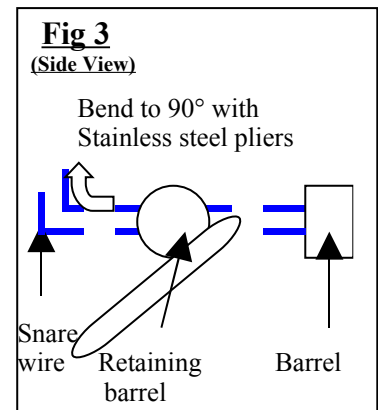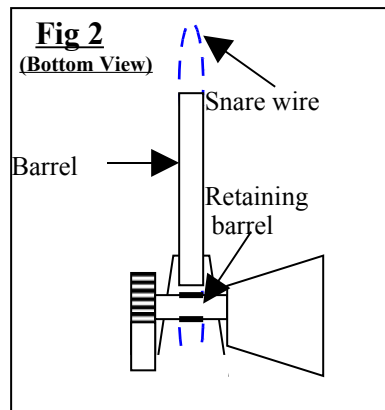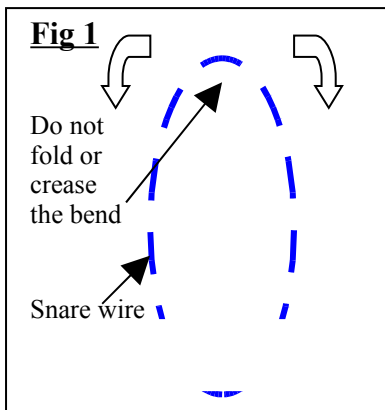

To disassemble the Foster's Enucleation Snare:

1. Depress the retaining clip and rotate the securing handle until the snare wire is flat and is in line with the lumen. If necessary use a pair of stainless steel pliers to carefully pull at the patient end of the wire – See Fig 2 and Fig 5.
2. Using a pair of stainless steel pliers carefully straighten both ends of the snare wires, if the ends are not straightened they may scratch the inside of the barrel as they are retracted.
3. Carefully pull the snare wire out from the handle using the stainless steel pliers – Fig 5.
4. It is recommended to dispose of the snare wire after use.