



ALTOMED

2 Witney Way, Boldon Business Park, Tyne & Wear. England. NE35 9PE.

Comments, queries or orders please contact:
Tel: (0) 191 519 0111
Fax: (0) 191 519 0283

Altomed Surgical Stool

Form ALT I016 – Issue 01/0602



Important read instructions carefully before use.
If in any doubt contact Customer Services at Altomed Limited.



Section 1 - Assembly:

Step 1.




Remove all the packaging, bubblewrap etc. Check that the following products are inside:
1 x Seat
2 x Armrests (if ordered)
1 x Backrest
1 x Alan Key

Step 2. Attach Arms

Step 3.


Step 4.

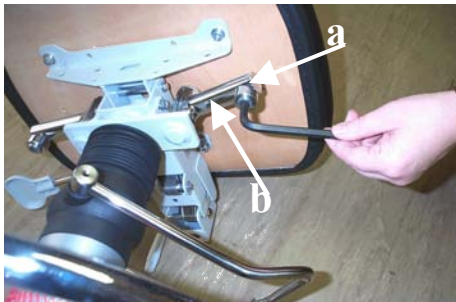

If the seat is supplied with arms remove the retaining bolts (a) from both sides of the supporting bar (b) as shown with the Alan Key.



Select one of the arms and slide it over the supporting bar (b).

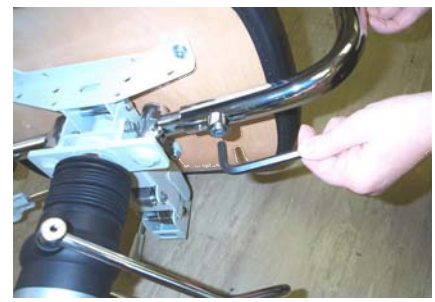

Secure the arm in position by re-inserting the retaining bolt. Tighten the arm into position using the Alan Key.


Step 5. Adjust Arms

Step 6. Attach Backrest


Step 7.



Loosen knob (c), adjust height of arms as required then tighten.

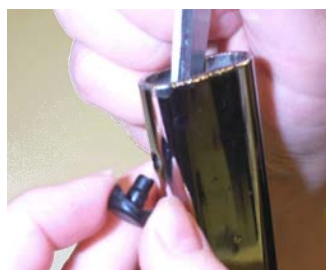

Remove retaining cap at base of seat by pushing it out using the flat end of a strong pair of scissors, screwdriver or similar.

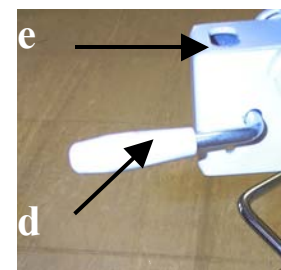

Ensure the Backrest Lever (d) is in the horizontal position

Step 8.

Step 9.

Step 10.



Insert the backrest into the Backrest Slot, (see e Step 7).



Replace the retaining cap that was removed in Step 6 above to secure the backrest.



Lock the backrest into position by setting the Backrest Lever (d) into a vertical position as shown

Step 11. Adjust Backrest



To adjust the position of the backrest, sit in the seat and pull up the Position Lever (f). Push the backrest into the position required. Lock off by letting go of the Position Lever.

Step 12. Adjust Height



To raise the height of the chair, pump the Height Lever (g) until the desired height is reached. To lower the height of the chair, push the Height Lever (g) right down until the desired height is reached. To lock the seat in position lift the Height Lever (g) up all the way until the chair locks securely.

Section 2 - Maintenance:

1. Keep the Alan Key in a safe place so that it does not get lost.
2. Dirt can be removed from the upholstery with a non-abrasive soft cleaner (i.e. soap). Never use abrasive or harsh chemicals to clean the stool.

Section 3 – Codes:

<u>Code</u>	<u>Description</u>
A 11020	Surgical Stool with a Waterfall seat
A 11022	Surgical Stool with a Saddle Seat
A 11024	Surgical Stool with a Waterfall Seat and Arm Rests (as pictured)
A 11026	Surgical Stool with a Saddle Seat and Arm Rests