



2 Witney Way
Boldon Business Park
Tyne and Wear
NE35 9PE
England

Parkin Lester Jones Tube Introducer

The following products are now available
from Altomed Limited.



To place an order contact Customer Services on
Tel: (0) 191 519 0111 Fax: (0) 191 519 0283

All Devices Are




Marked


This device is used to aid in the insertion of the Lester Jones Tubes during Conjunctivodacryocystorhinostomy.
For related devices ask Customer Services for a copy of the Original Lester Jones Tube (Promo04) and Cox System (Promo03) Promotional Sheets

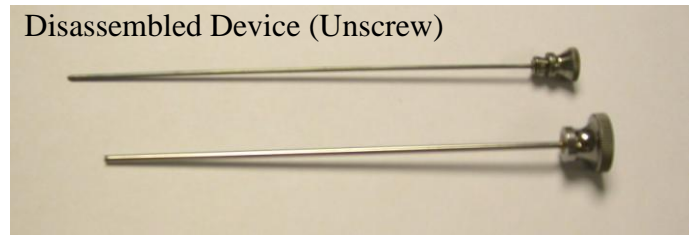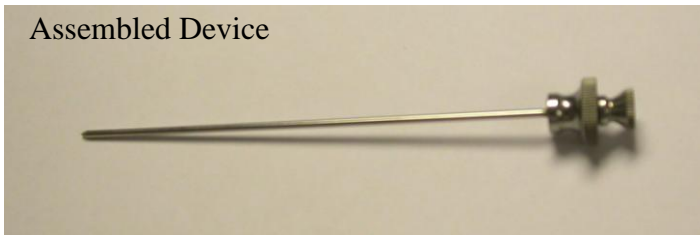


Directions For Use:

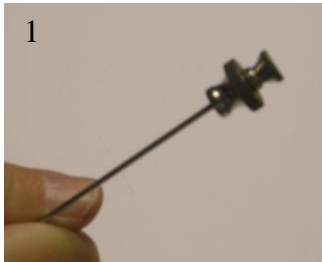


1. Insert the assembled device through the rhinostomy inferomedially into the nasal cavity creating a fistula.

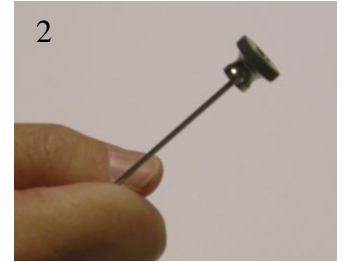

2. Unscrew the inner rod and carefully remove this from the outer body of the Introducer.

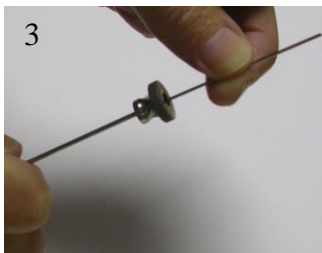

3. Feed a Small Insertion Rod (REF: COX05) through the lumen of the Introducer. Keep the Small Insertion Rod in place and then carefully remove the outer body of the Introducer.

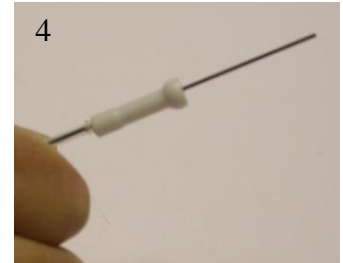

4. Feed the required size of Lester Jones Tube onto the small insertion rod. Using a Teflon Pusher to apply even pressure, carefully push the Lester Jones Tube into position then remove the Small Insertion Rod.

Warnings: The assembled device must be fed downwards, away from the eye, into the rhinostomy. This minimises any risk of damage to the globe. Care must also be taken to ensure excessive forces are not applied as this may cause the device to cross the nasal space and pierce the nasal septum and other intranasal structures including the turbinates. Only suitably trained and qualified Surgeons should use these devices. The above information is for guidance only. Contact Altomed for detailed cleaning and processing guidance covered on the Altomed Stainless Steel and Titanium Instruments Care Sheet.

Codes and Descriptions – Contact Customer Services for pricing and availability

COX01 – Sterilization tray with compartments
COX02 – Measuring Gauge with Small Insertion Rod
COX03 – Trocar
COX04 – Teflon (PTFE) pusher

COX05 – Small Insertion Rod, Curved Ends
COX06 – Large Insertion Rod, Curved Ends
COX07 – Flat Ended Measuring Rod Measures tube length only
A11940 – Parkin Lester Jones Tube Introducer