



2 Witney Way
Boldon Business Park
Tyne and Wear
NE35 9PE
England
Tel: 0191 519 0111
Fax: 0191 519 0283

All Devices Are



Marked

The Crawford Fascia Lata Stripper

The following products are now available from Altomed Limited.

For pricing or to place an order contact Customer Services on

Tel: (0) 191 519 0111 Fax: (0) 191 519 0283

A6368 - Crawford Fascia Stripper

Product Description:

Overall length approximately 355mm.



Instructions for Crawford Fascia Stripper

To release for operation, turn off-centre disc next to finger rings one-half turn. Cutting blade is then free to move for use.

To disassemble for cleaning, swing right finger ring out, releasing the channel clip, and pull blade straight back, lifting the cutting blade out of its operating groove.

After cleaning, sterilize in disassembled state.

To replace, return cutting blade into its channel, dropping the two pivot posts into the appropriate notches. Push blade forward and swing ring clip over channel.

For unabridged processing instructions contact Customer Services at Altomed Limited.